



Cervical Dilator

VAGINAL DILATOR

LARGE

SMALL

Description:

A vaginal dilator referred as Cervical Dilator is a Tube shaped device used for Stretching the cervix for Gynecological examination. The device surface is smooth and without any edges.

Intended use :

Its inserted in to cervix through vagina to dilate narrow cervical opening due to congenital defect. It can also be used during

gynecological examination, treatment or Surgical procedures.

Intended Population:

Woman

Precaution:

Never use force when inserting the dilator

Initially you may notice some blood spotting after using the dilators. But follow the advice of your doctor.

Steps :

- Use the dilator in an environment that is comfortable and ensures privacy.
- The recommended position to use the dilator is to lie flat on your back, or on your side. Your knees should be bent and legs slightly apart.
- The dilator should be inserted at an angle towards the back wall of vagina, as pressure to the front wall of vagina may cause pain and discomfort. Never use force.
- Once have positioned the dilator, gently move the dilator in a circular motion for 5 minutes.
- As you remove the dilator, gently push it in and out slowly and move it from side to side.



Storage condition :

Store in a dry place away from direct sunlight and sources of heat.

Warning :

Sterile unless the package is opened or damaged. The device must not be reused and must be destroyed after use as per regulations.

Shelf Life : 5 Years

MADE WITH PRIDE IN INDIA

SMB CORPORATION OF INDIA

(An ISO 14001:2015 / ISO 13485:2016 Certified Company)

Plot No. 156, GIDC, Umbergaon, Valsad, Gujarat 396 170, INDIA.

E-mail: sales@smbcorp.com | Website : www.smbcorp.com

Mfg Lic. No. : MFG/MD/2024/000136

EC REP Obelis S.A. Boulevard General Wahis 53,
1030 Brussels, Belgium. Tel: +(32) 2.732.59.54
Fax: +(32) 2.732.60.03 | E-mail: mail@obelis.net



Rev. 01, Dated 15.05.2024