
 SMB Corporation Of India	
Item	: IFU - Pozzi Tenaculum
Specifications	: 100 gsm Glossy Art Paper
Size	: L 70mm x B 216mm
 Four Colour Job	

Front

Back



Pozzi Tenaculum Forcep

How to Use

The Pozzi Tenaculum Forcep should be used by or under the supervision of a Physician / Gynecologist

The single use Pozzi Tenculum Forcep is made from Medical grade Polycarbonate meant for single use, thereby eliminating the risk of infections.

This Medical device having jaws ending in sharp inward- pointing hooks for precise tissue grasping and to optimize the traction of the cervix. Its thinner end offers the practitioner a better visibility and the locking mechanism facilitates gripping. This device is with ratchet mechanism



Intended use:

The Pozzi Tenaculum Forcep is used to hold the Cervix while insertion of the IUD and is also known as Cervix Holding Forceps.

Procedure:

- Peel off the pouch just prior to use the device
- Take out the device from the pouch
- Hold the device through handle
- Use the device as per required application / Place

Contraindication and Warning:

- This device is for singly use only therefore Do not reuse the device
- Never use the force to use the device

Storage Condition:

Store properly to protect from heat, direct sunlight, water and mechanical damage. Store under cool, dry condition and away from direct sunlight.

Precaution:

Do not use the device if package is open or damaged

Disposal:

Dispose the used device as per prevailing regulation for the handling of potentially infectious material.

Shelf Life : 5 years

1	2	3	4	5	6	7
LOT					STERILE	
Batch Code	Date of Manufacture	Use - by Date	Do not Reuse	Caution	Sterile and Method of sterilization "1"	Manufacturer
8	9	10	11	12	13	14
CE 2862	Keep Dry	Do not use if package is damaged and consult instructions for use	Keep away from sunlight	Do not re sterilize	Temperature Limit	UDI
CE mark	Keep Dry	Do not use if package is damaged and consult instructions for use	Keep away from sunlight	Do not re sterilize	Temperature Limit	UDI
15	16	17	18	19	20	
MD	EC REP	Consult Instructions for Use	Single sterile barrier system	presence of phthalates (DEHP)	Medical Substance	
Medical Device	Authorized Representative in the European Community	Consult Instructions for Use	Single sterile barrier system	presence of phthalates (DEHP)	Medical Substance	

MADE WITH PRIDE IN INDIA

Legal Manufacturer :

SMB CORPORATION OF INDIA

(An ISO 14001:2015 / ISO 13485:2016 Certified Company)
13, 33-36 Prem Industrial Estate, Subhash Road, Jogeshwari (E),
Mumbai 400060, INDIA.

Tel.: +91-22-2824 6922 / 2834 5637, Fax.: +91-22-28215157

Email: sales@smbcorp.com; Website: www.smbcorp.com

Manufacturing Site :

Plot No. 156, GIDC, Umbergaon, Valsad 396170, Gujarat, INDIA.

Mfg. Lic. No.: MFG/MD/2024/000136

EC REP Obelis S.A, Boulevard Général Wahis 53, 1030 Brussels, Belgium
Tel: +(32) 2.732.59.54 • Fax : +(32) 2.732.60.03 • E-Mail : mail@obelis.net



Rev. 00, Dated 15.04.2025 QA/PZT/IFU