

SMB Corporation Of India

Item : SMB SPECULUM IFU

Specifications : 100 gsm glossy art paper

Size : L 140 mm x H 180 mm

Pantone 218 C

Four Colour Job

Front


Back

SMB
Adjustable Vaginal / Réglable Vaginal / Vaginites Ajustables

SPECULUM
SPÉCULUM / ESPÉCULO
Medium / Small

Description :

A transparent speculum is a duck-bill-shaped device made up of clear, high strength, non-toxic polymer used for gynecological examination. It is Designed to eliminate the time and expense of reprocessing metal specula. Single use speculum also minimizes the risk of cross-contamination associated with improper cleaning and sterilization. Locking facility enables necessary vaginal widening adjustments. It facilitates self-holding in a variety of positions. Viewing of the cervix and vagina transparent plastic speculums providing good visibility to the practitioner and greater comfort to the patient than metal speculums, and preventing thermal shocks.

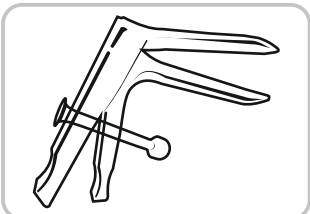

Intended Use :

The speculum is used for vaginal exams by health care professional/gynecologist. They use it to open the walls of the vagina and examine the vagina and cervix. It is a single use device.

It is a Gamma sterile, Single Use transparent Medical device

Application domain: vaginal examination

Uses of speculums

There are several reasons why healthcare professional/gynecologists might use a speculum. They Are as follows:

■ Pelvic exam

Gynecologists do a **pelvic exam** to check the vagina, cervix, and other reproductive organs for problems. The speculum opens up the vaginal canal to make these organs easier for the doctor to see.

During this exam, the doctor might use a special brush to remove a few cells from the cervix. This is called a **Pap test or Pap smear**. The cells go to a lab, where they're checked for early signs of cervical cancer.

Cell samples can also be removed from the vagina and cervix to check for sexually transmitted diseases (STDs).

Gynecologist also use a speculum for procedures like these:

- Vaginal hysterectomy. This procedure removes the uterus through the vagina.
- Dilation and curettage. This procedure opens (dilates) the cervix and removes part of the uterus lining.
- Intrauterine insemination (IUI) and in vitro fertilization (IVF). These procedures help women with fertility problems get pregnant.
- Intrauterine device (IUD) placement. IUDs are a reversible form of birth control that is placed inside of the uterus.

Complications related to speculum use :

Pelvic exams can be a little uncomfortable when your doctor places the speculum inside your vagina and opens it. However, there are few risks involved as long as the speculum is sterile. If it hurts, you can ask the doctor to use a smaller speculum.


The speculum might feel like it's stretching out your vagina, but it only opens up the vaginal canal temporarily. It won't widen or loosen your vagina. A speculum shouldn't cause damage or injury when used by a trained doctor.

Storage Conditions:

Store in a dry place away from direct sunlight, sources of heat, water and mechanical damage.

1	2	3	4	5	6	7
Lot No./ Batch No.	Manufacturing Date	Insert Before Date	Do not Reuse	Warning	State and Method of sterilization "G" and Sterilization by Radiation Method	Manufactured by
8	9	10	11	12	13	14
CE mark	Protect from rain or water	Do not use if carton is damaged	Protect from Sun light	Do not re sterilize	Temperature Limit	UDI

MADE WITH PRIDE IN INDIA



SMB CORPORATION OF INDIA
(An ISO 14001:2015 / ISO 13485:2016 Certified Company)

Unit - I : 13, 33-36, Prem Industrial Estate, Subhash Road, Jogeshwari (E), Mumbai 400 060, INDIA.

Unit - II : Plot 156, GIDC, Umbergaon, Valsad, Gujarat 396 170, INDIA.

E-mail : sales@smbcorp.in

Website : www.smbcorp.in

Rev. 00, Dated 15.02.2022