

 SMB Corporation Of India	
Item	: IFU - Thread Cutting Scissors
Specifications	: 100 gsm Glossy Art Paper
Size	: L 70mm x B 216mm
 Four Colour Job	

Front




IUD Thread Cutting Scissor

How to Use

The IUD Thread scissor should be used by or under the supervision of a Physician / Gynecologist

The single use IUD Thread cutting Scissor is made from Medical grade Polycarbonate meant for single use, thereby eliminating the risk of infections.

This (IUD Thread Cutting scissor) is a specially designed instrument having embedded blade at the end (Opposite to gripping system).



Intended use:

The device is used to cut the extra length of sutures / threads of IUDs. It is also known as Suture Cutting Forceps.

Procedure:

- Peel off the pouch just prior to use the device
- After placing the IUD in the uterus, enter the device leading front side in to the vagina
- Cut the extra length of the thread
- Gently pull out the device

Back

Contraindication and Warning:

- Do not reuse the device
- Never use force to enter the device in the vagina
- Smoothly Pull out the Device

Storage Condition:

Store properly to protect from heat, direct sunlight, water and mechanical damage. Store under cool, dry condition and away from direct sunlight.

Precaution:

Do not use the device if package is open or damaged


Disposal:

Dispose the used device as per prevailing regulation for the handling of potentially infectious material.


Shelf Life : 5 years


1	2	3	4	5	6	7
Batch Code	Date of Manufacture	Use - by Date	Do not Reuse	Caution	Sterile and Method of Sterilization "1"	Manufacturer
8	9	10	11	12	13	14
CE mark	Keep Dry	Do not use if package is damaged and consult instructions for use	Keep away from sunlight	Do not re sterilize	Temperature Limit	UDI
15	16	17	18	19	20	
Medical Device	Authorized Representative in the European Community	Consult Instructions for Use	Single sterile barrier system	presence of phthalates (DEHP)	Medical Substance	

MADE WITH PRIDE IN INDIA

Legal Manufacturer :
 **SMB CORPORATION OF INDIA**
 (An ISO 14001:2015 / ISO 13485:2016 Certified Company)
 13, 33-36 Prem Industrial Estate, Subhash Road, Jogeshwari (E), Mumbai 400060, INDIA.
 Tel.: +91-22-2824 6922 / 2834 5637, Fax.: +91-22-28215157
 Email: sales@smbcorp.com; Website: www.smbcorp.com

Manufacturing Site :
 Plot No. 156, GIDC, Umbergaon, Valsad 396170, Gujarat, INDIA.
Mfg. Lic. No.: MFG/MD/2024/000136

 Obelis S.A, Boulevard Général Wahis 53, 1030 Brussels, Belgium
 Tel: +(32) 2.732.59.54 • Fax : +(32) 2.732.60.03 • E-Mail : mail@obelis.net



Rev. 00, Dated 15.04.2025 QA/TCS/IFU